



FLÄCHEN-
DESINFIZIERUNGSMITTEL

Fugaten®-Spray (Lysoform®-Spray)

Gebrauchsfertige Sprühdesinfektion auf Alkoholbasis
zur Schnell-Desinfektion für Flächen, Gegenstände
und Medizinprodukte

Désinfectant en spray à base d'alcool pour
des petites surfaces, objets et dispositifs
médicaux. Prêt à l'emploi.

Alcohol based ready-to-use disinfecting
spray for a rapid disinfection of surfaces,
objects and medical devices



CE 0482

Instructions for use - Please read carefully!

Fugaten[®]-Spray (Lysoform[®]-Spray)

Ready for use

Disinfection-Spray based on alcohol for rapid disinfection of medical devices

Composition

100 g **solution** contains: 70 g Ethanol (denatured)

Range of application

For rapid disinfection of

- Small surfaces and objects (e.g. operating tables, examination chairs etc.)
- Surfaces difficult of access
- All alcohol resistant materials
- Product exists also without perfume for ex. for use in the food industry

Mode of application / dosage

Use undiluted only!

Spray **Fugaten[®]-Spray** on objects and surfaces from about 30 cm (max. 2 m²) until they are fully wet (max. 50 ml/ m²).

Disinfection of surfaces

in hospital and doctor's surgery concentrated – 1 min

virucidal against enveloped viruses*

incl. HBV/HCV/HIV,

Rota and Adeno virus

concentrated – 30 sec

virucidal*

concentrated – 15 min

* according to the recommendations of Robert Koch Institute Berlin / Federal Health Gazette 01/2004

Warnings / storage

- **Fugaten[®]-Spray** is easily inflammable (flash-point according to DIN 51755: 19,5°C)!
- Do not spray in an open flame and/or on incandescent substances.
Max. quantity to be sprayed: 50 ml/ m²
- Observe directions for use and safety rules (see „Safety rules for the prevention of fire and explosions caused by disinfectants based on alcohol“ of the Professional Association for Health Care and Welfare Services, Hamburg).
- Keep disinfectant out of reach of children, away from ignition sources, and store at 5 - 30 °C.
- Do not use beyond expiry date.

Properties

- Bactericidal (incl. tb), fungicidal, yeasticidal
- Rapid action and drying after spraying on surfaces and objects
- Freshly scented; deodorizes the surrounding air
- Good wetting power
- No adverse effect on most materials
- Ready to use product in refillable, conical polyethylene bottles with spray-pump

Special precautions

a) Counter-measures in case of improper use

Fugaten[®]-Spray is irritating in contact with the mucous membranes (eyes, throat, stomach, bowels etc.). In case of such irritations dilute product immediately. If it has been taken **orally**, make patient **drink plenty** of water; if it has been in contact with the eyes, rinse thoroughly with **plenty** of water. In each case, get medical attention.

b) Miscellaneous

Do not spray on materials which are not compatible with alcohol (coated, pickled surfaces, Plexiglas etc.).

c) Use disinfectant safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

Translated by Lysoform Dr. Hans Rosemann GmbH, Berlin - Germany

Standard packaging/transport units:

10 x 500 ml bottle with spray pump
12 x 1000 ml bottle with spray pump
12 x 1000 ml bottles
2 x 5 litre canister
2 x 5 litre canister (unscented)

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

Lysoform Desinfektion AG
Burgstraße 28, CH – 8750 Glarus
Tel.: +41-55 640 62 79 **0049-12**