


GERÄTE/ZUBEHÖR

Lydos F-Wandspender für 500 ml Flaschen

Spender für flüssige Handwaschmittel und
alkoholische Händedesinfektionsmittel

Dispenser for liquid soaps and
alcoholic hand disinfectants

for 500 ml bottles

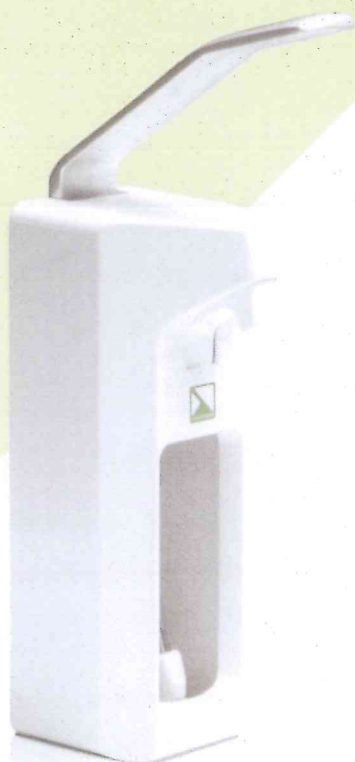

LYDOSFORM

CE

Instructions for use - Please read carefully!

Lydos F – Wall Dipenser

For 500 ml - bottles

Dispenser for liquid soaps and hand disinfectants based on alcohol

General advice:

It is recommended to use the dispenser with those products only that are compatible with the material the dispenser is made of.

The dispenser should be occasionally cleaned with warm water and disinfected with disinfectants that are based on alcohol.

Resets of products do not dry and stick together if the container of the preparation is always filled. In case of blockage please clean with hot water and brush only. Do not unscrew the valve or open the pump!

- 1) Dispenser for liquid hand soaps and hand disinfectants based on alcohol.
- 2) The housing is fixed to the wall plate by clips. Take the housing from the wall plate by pulling forward.
- 3) Installation: in a suitable height with the 4 screws and wall plugs that are enclosed.
- 4) Place the bottle in position and pump several times.
- 5) If necessary: Adjust the dosing limit. Therefore, turn the dosing aid made of plastic in the desired position.
- 6) Fix the housing with the clip onto the wall plate.
- 7) Hygienic elbow or forearm-tripping.

Packaging

Lydos F wall dispenser

5810

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

Lysoform Desinfektion AG
Burgstraße 28, CH – 8750 Glarus
Tel.: +41-55 640 62 79

0028-3